


Stretch Film



Tapes



Safety

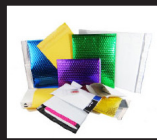

Poly Bags,
Mailers & PLE's



Labels



"Your Source for Packaging, Shipping, & Safety Supplies"

www.wraptite.com Distributors Only

An ISO 9001:2015 Certified Company






Intertek

Thermal Imager Quick Install Guide

Imager:

- **Power** - Open the port cover on top of the imager, connect the included USB-C cable to a computer or outlet to charge, once charged, hold the Yellow power button to turn on.
- **Language** - Scroll to the settings menu, it should look like a gear. Select it then select the first option and choose English.
- **Temp. Unit** - While in the settings menu, select the option labeled "Temperature Unit" and change to Fahrenheit.
- **Alarm Settings** - Also in the settings menu, select the option labeled "HI/LO Alert" to set the threshold for the alarm.
- **Image Mode** - Exit the settings menu and scroll to the left for the Image Mode settings, select this to toggle between Digital, Thermal, and Fusion modes.
- **Tripod** - For Tripod mounting, refer to the video found here www.wraptite.com/healthcare/thermometer-thermal-imagers/thermal-imagers.html on our website.
- **Software** - The software to the imager is found here, www.wraptite.com/nctipfhf01.html once installed just launch BEFORE connecting imager via USB.
- **Temp. Taking** - Setup the Imager at desired location and readings will automatically show.

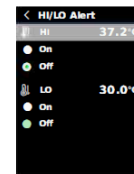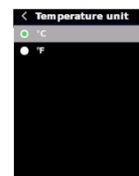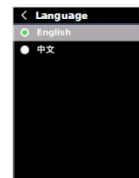

www.wraptite.com



888.Wraptite (972.7848)
www.wraptite.com

PHONE: 440-349-5400

FAX: 440-349-5432

info@wraptite.com

Visit www.wraptite.com to see more products and tons of special offers.